

# Vapmite

This new, affordable, vaporiser requires a 12v battery, such as a car battery to provide the power to heat the oxalic acid crystals e.g. Api-Bioxal. You will also need a piece of foam for the entrance.

Fit and adjust the cross piece of the Vapmite using the jubilee clip provided. It should be so adjusted that when placed in the floor of the hive the pan does not touch the floor, but is inclined slightly upwards, avoiding the bottom of the frames.

Place 2.3g of the product in the shaped pan at the end of the shaft (one level scoop full). Insert into the hive entrance so it is approximately 2/3 of the way in to the hive. Seal the entrance around the Vapmite shaft with the foam.

Fit the clamps to the battery terminals, red to + and black to -. Vaporising the product will take approximately 3 minutes. Remove one of the battery clamps to switch off the Vapmite.

Remove foam and vaporiser, replace foam and leave hive sealed for 15 minutes.

Clean the Vapmite with water once it has cooled down completely.

## WARNING

- **Do not leave on for more than 3 minutes per treatment/hive**
- This device gets very hot very quickly. Wear suitable protective clothing.
- Never stand downwind of vaporising product.
- Be aware that handling any product used in the Vapmite must be done with care.
- Skin, eye and respiratory tract are all susceptible to severe damage if the product is not handled correctly. Wear suitable protective clothing, goggles and face mask (at least type FFP2).
- Follow the manufacturer's instructions on the product in use.



**E.H. THORNE (Beehives) LTD**

Beehive Business Park, Rand, Nr. Wragby, Lincolnshire LN8 5NJ

Tel. 01673 858555

sales@thorne.co.uk      www.thorne.co.uk